


# Waste Disposal while Traveling

## When you travel, waste disposal is your responsibility

- Leftover solution in solution bags and effluent in drain bags and solution lines should be drained, then disposed of in accordance with local laws
- This typically means these products are thrown away with regular garbage
- Some areas are able to recycle materials; you should check recycling websites at your destination

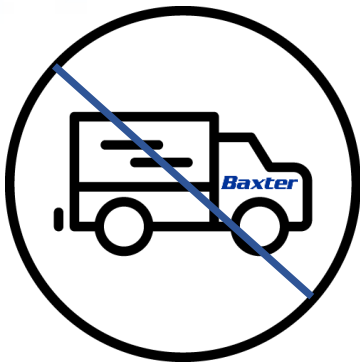

At the end of your trip, you may have leftover PD supplies; you can either take them home with you or dispose of them  
Baxter cannot accept returns of unused supplies

- Unused disposable products like APD sets, caps, etc. should be disposed of in accordance with local laws, which usually means they are thrown away with regular garbage
- Unused solution bags should be drained first, then disposed of in accordance with local laws
- To drain, break the seal of the solution bag, uncap the connector, and place the bag in the sink, bathtub, or shower to allow it to drain
  - Alternatively, you may cut the bag and drain the solution

Baxter will not pick up unused product. It is your responsibility to discard of any used or unused product from your travels.